

BONE INJECTION GUN

INTRASOSSEOUS INFUSION

DISPOSABLE · AUTOMATIC
INTRASOSSEOUS
INJECTOR

Intraosseous Access in Seconds



Adult Depth is Pre-Set



Pediatric Depth is Adjustable by Age in Years



1 Position the B.I.G. 90° to the surface



2 Remove safety latch



3 Trigger



4 Remove B.I.G. with side-to-side motion



5 Clip & Secure



6 Remove trocar needle



7 Connect extension set or tubing



8 Aspirate / Infuse*



* Flush 10-20cc in Adults / 5-10cc in Pediatrics
* Bone marrow may not always be present.

